



Installation Instructions

**select series
surface mount consoles**



note

Console installation should conform to local building codes

The installation instructions provided are for the Amico standard products. Slight modifications to the installations may be required. The product shipped to the job site may not be exactly as shown on the installation drawings.

Before Installation


- 1 Review a copy of the final approved shop drawing(s) and/or submittal package. These documents will provide you with technical information specific to your installation, such as:
 - equipment types & quantities
 - room numbers and location
 - shop drawings of each unit type
- 2 Locate the console unit from the shipping boxes that you wish to install. The contents of each shipping box are listed on the shipping document(s).

Installation


- 1 Remove all electrical and gas cover plates by loosening any screws. Place everything in a safe and clean location.



- 2 Remove the fascia by loosening the 2 screws (1 on each side) attaching it to the console. Place everything in a safe and clean location.

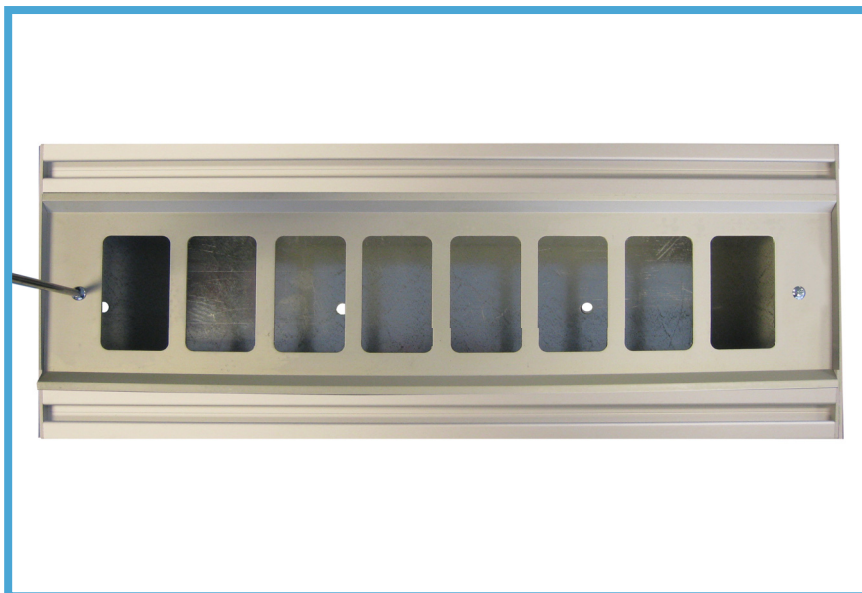


- 3 Remove any aluminum device covers that are blocking access to the back plate. This is done by loosening the screws (typically 2 per cover) attaching it to the console box. Place everything in a safe and clean location.

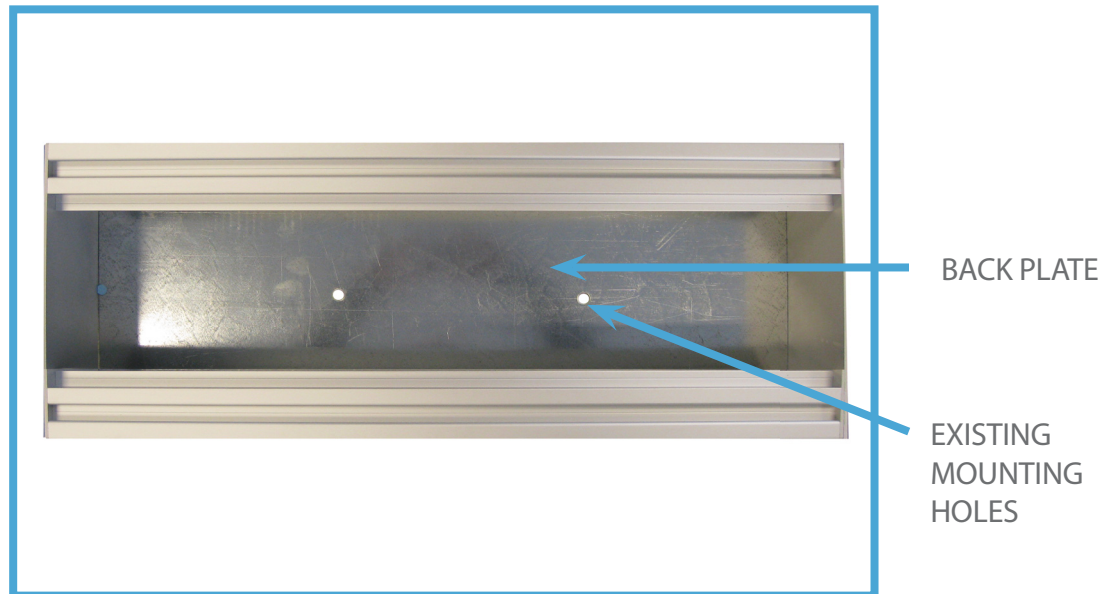

- 4 Secure the console to the wall structure. Use existing holes if it matches the stud locations or drill new mounting holes as necessary (pictured above). Minimum two mounting locations should be used.

- 5 Make all necessary electrical medical gas connections.

- 6 After connections are made, reassemble console in the reverse order as shown in the steps outlined above.

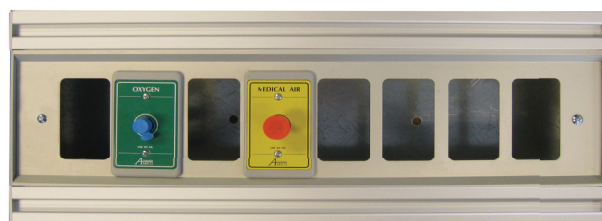



85 Fulton Way, Richmond Hill
Ontario, L4B 2N4, Canada

14 Madison Road, Fairfield
NJ 07004, USA

Toll Free Tel: 1.877.462.6426
Toll Free Fax: 1.866.440.4986
Tel: 905.764.0800 | Fax: 905.764.0862
www.amico.com



US LISTED

MAY 2008